





Your kit includes...

- 1 x Alcohol Free Cleansing Wipe
- 1 x Plaster

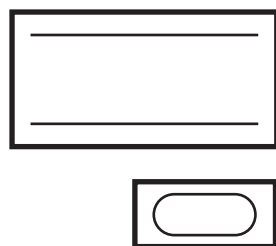

- 1 x Customer Details Form
- 1 x Prepaid* Envelope

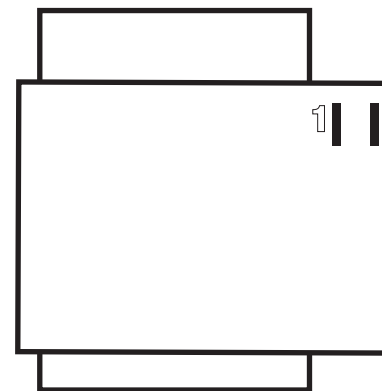

*UK and Ireland Only

Instructions for use




Lorisan Ireland
5 Fitzwilliam Terrace
Bray
Co. Wicklow
Ireland
(ROI) 01 20 22 701
www.lorisan.ie


Lorisan
Genesis 3
Church Lane
York
YO10 5DQ
United Kingdom
+44 (0)1904 428 550
www.lorisan.com

- 2 x Single-Use Lancets

(Do not use if the protective cap has been previously removed)

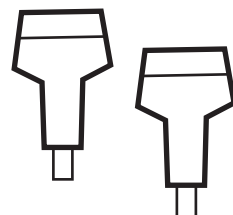

- 1 x Absorbent Wand and Container

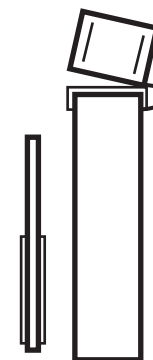

YorkTest have assembled this procedure pack according to Article 12 of the Medical Device Directive 93/42/EEC. All individual medical devices (lancets, plaster, wipe, absorbent wand and container) possess CE marking and are used according to their indications for use.

Companies Registration Number: 03570476
Lorisan is a registered trade mark ©Lorisan 2014
LOR1432 v7

Caution

- Keep out of reach of children
- Adult supervision is required for children between 2 – 18 years old
- The lancet should not be shared with others
- Use caution when handling blood
- Return lancets in the kit box provided for safe disposal

Do not take this test if you are:

- Suffering from a blood borne disease such as Hepatitis or HIV
- Under the age of 2
- Pregnant or breast feeding

If you have any difficulties collecting the sample don't worry, please call our Customer Care team on **+44 (0)1904 428 550 (ROI) 01 20 22 701**



If you have any difficulties collecting your sample please call our Customer Care team on
+44 (0)1904 428 550
(ROI) 01 20 22 701

- 1** Complete the Customer Details Form and sign it



- 2** Write your name and the date on the wand container



- 3** Wash your hands in warm water and dry to encourage circulation



- 4** Wipe the top of your finger with the Alcohol free cleansing wipe and allow to dry



- 5** Twist and pull the cover from the lancet



- 6** Place the tip of the lancet on your finger and press the yellow part of the lancet down to produce a large droplet of blood




- 7** Stand up, hold your hand below your waist and massage your finger towards the pin prick to encourage blood flow



- 8** If you have difficulty in producing enough blood try repeating Step 6 on another finger with the second single-use lancet



- 9** Hold the wand by the plastic cover and dip into the blood and allow to draw up

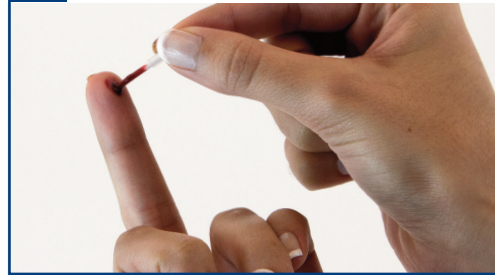

- 10** Slide the plastic cover along the wand and rotate in order to completely soak the wand



- 11** It is important that the wand is fully saturated as this contains the precise amount of blood required to be tested



- 12** Place the saturated wand back into its container. Apply the plaster to your finger



- 13** Return the sample and both lancets to the kit box



- 14** Enclose the signed Customer Details Form and the kit box in the prepaid* return envelope and post to Lorisian immediately



*UK and Ireland Only

LORISIAN

Remember to include your signed Customer Details Form, we cannot test your sample without it.

+44 (0)1904 428 550
(ROI) 01 20 22 701

